


1. Establish that the patient is in cardiopulmonary arrest (the patient must be unresponsive, not breathing normally and showing no signs of circulation).
2. Press ON/OFF to turn on the defibrillator (the green LED illuminates). Voice prompts will sound, guiding you through the rescue process.
3. Prepare the patient for therapy electrode placement. Remove clothing from the patient's upper torso.
4. Open the therapy electrode packet and remove the electrodes. Slowly peel back the protective liner on the electrodes, beginning with the cable connection end. Safely discard the liner to prevent slipping.
5. Apply the therapy electrodes to the patient's chest. Slowly peel back the protective liner on the electrodes, beginning with the cable connection end. Safely discard the liner to prevent slipping.

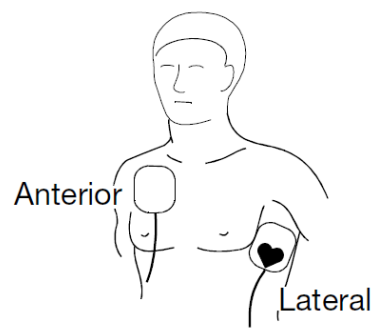

6. Follow the voice prompts and messages.